Continuous Glucose Monitoring System Quick Start Guide in English

1 Setting up App

Secure internet access is required during setup

- 1. Get Started**
 - Download the iCan CGM App from App Store or Google Play.
 - Open the App
 - Login or create an account
- 2. Set up**
 - Follow App instructions to setup
 - For instructions on inserting the sensor, use **Inserting the Sensor** below
- 3. Sensor Warmup**

During the 2-hour Sensor warmup, you'll not get any readings or alerts until the warmup is done.
- 4. Understand Your CGM Results**

Go to the iCan User Guide to find out more information.


2 Inserting the Sensor

- 1. Get Ready**
 - Wash your hands with soap and water
 - Choose an insertion site on the abdomen (**Caution:** Avoid scars, moles, stretch marks, lumps and insulin injection sites. To prevent skin irritation, rotate sites between applications. Consider your comfort and activities.)
 - Clean the insertion site with alcohol pads. (**Caution:** Let the alcohol dry before insertion.)
- 2. Pair with the Transmitter**
 - Pull out the Transmitter Pack from the bottom of the Sensor Pack box.
 - Check the first 8 digits SN codes of Sensor Pack and Transmitter SN codes to match with each other before pairing your Sensor with the Transmitter.
 - Open your App and make sure to turn Bluetooth on. Scan the QR Code on the Sensor Pack using your App
- 3. Apply the Sensor**
 - Peel lid completely off the Sensor Pack
 - Place the Transmitter Pack on a hard, flat surface. Line up the blue mark on the Sensor Applicator with the mark on the Transmitter tray.
 - Firmly press the Sensor- Applicator down into the tray until you hear a click and it comes to a stop.
 - Gently turn the safety switch from the "Locked" icon to "Unlocked" icon until you hear a loud click and can not turn further (**Caution:** Do not push down on the middle button until the safety switch is fully released to prevent unintended results or injury)
 - Gently pull the armed Sensor-Applicator straight up. (**Caution:** Do not touch the adhesive tape)
 - Now the Sensor-Applicator is ready for insertion.
- 4. Insert the Sensor Applicator**
 - Place the armed Sensor-Applicator over the chosen site
 - Press the button in the middle until you hear an injection sound
 - Gently pull the Applicator away from your body (**Caution:** Discard the used Applicator according to your local regulation.)
 - Press the adhesive around the sensor so that it sticks securely to your skin (**Caution:** Check your sensor site for bleeding after you insert the sensor. If there is bleeding that does not stop, remove the Sensor, apply steady pressure using sterile gauze until the bleeding stops.)

3 After 15 Days, End Session

- 1. Remove the Sensor**
 - Pull up the edge of the adhesive.
 - Slowly peel the adhesive away from your skin in one motion.
- 2. Disposal**

Discard the used sensor according to local regulation.

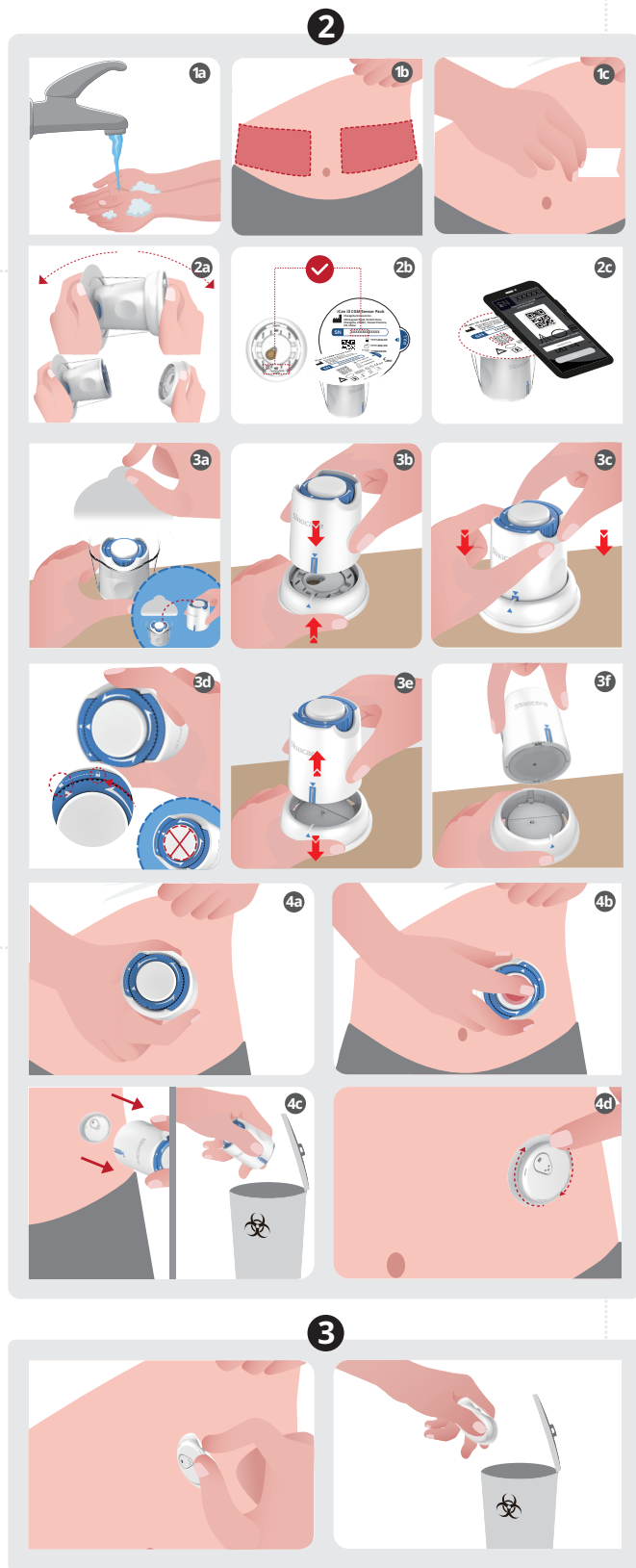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All graphics are representational. Your product may look different

Review full System instructions and Safety information in the iCan User Guide.

The User Guide is provided in electronic form. Please visit iCan-cgm.com website to download a PDF.

If you need a hard copy of the User Guide, please contact the dealer or send the request to the email: support@icancgm.com. It will be provided within seven days for free.

If you have any questions, please send the email to support@icancgm.com

Contact Us:

Email: support@icancgm.com

Website: iCan-cgm.com